

PRECLEANING

PRE-FOAM®

LONG LASTING FOAM PRE-SOAK CLEANER FOR ENDOSCOPES & MEDICAL EQUIPMENT

PreFoam is a new neutral, detergent for use as a pre-soak cleaner. It is specifically formulated to apply to medical devices, instruments and equipment as a stable long lasting foam that immediately commences to break down blood and bio-burden contaminants. This prevents the contaminants from drying out on the devices thus improving the effectiveness of the subsequent cleaning, speeds up the overall cleaning process, whilst eliminating and preventing the formation of biofilm.

Its formulation is effective on all types of protein and organic material and is suitable for use with surgical and endoscopic equipment and components, laryngoscopes, biopsy forceps, haemostats, etc.

PreFoam is registered as a Class I medical Device



Primary features attributable to Pre-Foam are:

- Is suitable for application to proteinaceous and organic contaminants from sensitive instrument materials
- Prevents blood, debris and body fluids from drying on instruments and medical devices
- Eliminates the formation of biofilm
- Environmentally friendly and Free-rinsing
- Increased speed and efficiency
- Improves device cleaning process
- Decreases turn round time

Packaging

- 750ml Trigger Spray

PreFoam is a new neutral, detergent for use as a pre-soak cleaner. It is specifically formulated to apply to medical devices, instruments and equipment as a stable long lasting foam that immediately commences to break down blood and bio-burden contaminants.

PRECLEANING

Application

Directions for Use

1. Spray directly on soiled instruments and scopes immediately after use.
2. Allow instruments to soak in foam until ready for cleaning. Apply more as needed to keep instruments and scopes moist.
3. Rinse instruments before cleaning
4. Discard used solution

PRE-FOAM®
LONG LASTING FOAM PRE-SOAK CLEANER FOR
ENDOSCOPES & MEDICAL EQUIPMENT

Pre-Gel Properties

- Pre-Foam is not flammable, and biodegradable.
- Shipping class is non-hazardous.

See Material Safety Data Sheets for details.

| | |
|--|-------------------------|
| Appearance | Clear Colourless liquid |
| Odour | Slight soapy odour |
| pH | 9.8 units (Neat) |
| Density g/cm³ @ 20°C | 1.00 (neat) |
| Solubility in water | Completely miscible |
| Flashpoint (Abel closed cup) | None |



Amity Limited, Libra House, West Street,
Worsbrough Dale, Barnsley, South Yorkshire,
S70 5PG, UK

T: +44 (0) 1226 747660

F: +44 (0) 1226 291936

E: sales@amityinternational.com

WWW.AIHC.CO.UK

© Amity Limited all rights reserved 2018

DISTRIBUTED BY: